

CleanGuide™ Disposable OTW Esophageal Dilators



CONMED CleanGuide™ Disposable OTW Esophageal Dilation System			
Part Number	Diameter (D) - mm	Diameter (D) - Fr	Taper Length (L1) - cm*
CG-05-3	5mm	15Fr	13cm
CG-06-3	6mm	18Fr	13cm
CG-07-3	7mm	21Fr	18cm
CG-08-3	8mm	24Fr	18cm
CG-09-3	9mm	27Fr	16cm
CG-10-3	10mm	30Fr	16cm
CG-11-3	11mm	33Fr	16cm
CG-12-3	12mm	36Fr	16cm
CG-13-3	13mm	39Fr	16cm
CG-14-3	14mm	42Fr	16cm
CG-15-3	15mm	45Fr	16cm
CG-16-3	16mm	48Fr	16cm
CG-17-3	17mm	51Fr	16cm
CG-18-3	18mm	54Fr	16cm
CG-19-3	19mm	57Fr	16cm
CG-20-3	20mm	60Fr	16cm
3 X 'D' mm = 'D' Fr			
*This length is an approximation and may be within +/- 1cm			

How to Use the Measurement System

Step 1: Using the endoscope, measure the distance from the dental arch to the stricture (**value 'X'**)

Step 2: While dilating, identify the dilator's position in reference to the dental arch (**value 'Y'**)

** When $X = Y$, the dilator's tip has reached the stricture

** When $X + L1 > Y$, the dilator's max outer diameter has reached the stricture